



MICRO PLUS – USER MANUAL

1. Device Overview

MICRO PLUS is a professional electrotherapy and body-contouring system using galvanic current for facial and body applications.

2. Device Modes

BURST – Lymphatic drainage, toxin and edema removal

NORMAL – Blood circulation, cellulite and fat breakdown

MOD – Tightening, lifting, and firming

3. Frequency Settings

Pulse Width: 260

Pulse Rate: 150 Hz

4. Application Areas

Face, Abdomen, Legs, Back, Arms, Chest

5. Benefits

- Improves blood circulation
- Stimulates collagen & elastin
- Tightens muscles
- Reduces cellulite
- Activates lymphatic system



6. Session Durations

Legs: 40–60 min

Abdomen: 20–40 min

Back: 20–40 min

Chest: 20–40 min

Arms: 20–30 min

Face: 30–40 min

7. Contraindications

Not suitable for pregnant women, cancer patients, metal implants, kidney disease, organ transplant patients.

8. Cleaning & Safety

- Disinfect body heads with alcohol
- Face heads cleaned with wipes + UV sterilization
- Avoid lymph nodes and acne areas

9. Technical Specs

Output Channels: 4

Max Current: 100 mA

Power: AC 220–240V

LEVENT MAH.ALTZEREN SOK NO:29 LEVENT-İSTANBUL

Cep:0532 6328360